

The DizzyFIX device is safe, simple to use, reliable and effective.

### Research:

Three clinical trials have been conducted at the University of Western Ontario regarding the DizzyFIX device. The concept of a head-worn visual feedback device has been proven with healthy volunteers in a randomized clinical trial. In addition, the reliability of the device in predicting a correct maneuver has been demonstrated. Finally, 92% of patients who used the DizzyFIX for the self treatment of BPPV were successful. This cure rate did not differ significantly from the 87% success rate in patients who had treatment maneuvers guided by an ENT physician. These three studies have found the DizzyFIX device to be safe, simple to use, reliable and effective.



For more information about this or other products from Clearwater Clinical Limited please contact us:



+ Manufactured by  
Clearwater Clinical Limited™  
100 - 111 5th Ave SW, Suite 258  
Calgary AB, T2P 3Y6  
CANADA  
1-877-349-9934

Vendor Contact Information

### ★ ★ ★ ALL STAR SALES

All Star Sales  
4 Vata Court Unit #15  
Aurora, Ontario, L4G 4B6  
1-800-361-0072  
info@allstarsales.ca

Copyright 2005-2007 Clearwater Clinical Limited™

[www.allstarsales.ca](http://www.allstarsales.ca)

DFPTB-01-20070601-01 PDF



Anyone can treat dizziness\*  
with the DizzyFIX



\*The DizzyFIX treats dizziness related to BPPV



## Everyone has experienced dizziness.

The most common cause of dizziness related to the ear is Benign Paroxysmal Positional Vertigo (BPPV). BPPV is caused by free-floating crystals in the inner ear. These crystals cause spinning dizziness when disturbed by head motion.

There is, however, an effective treatment for BPPV. The DizzyFIX enables anyone to perform a "Particle Repositioning Maneuver". This maneuver cures most cases of BPPV.

Whether being used by a patient at home, a doctor or other health professional, this device helps diagnose and treat BPPV.


## + Dizziness can be debilitating.

Benign paroxysmal positional vertigo (BPPV) is the most frequent cause of peripheral vertigo. The incidence of BPPV increases with age and has been reported as greater than 9% in people over 60 years old. Recurrence rates have been reported in up to 50% of patients.

## Do you have the symptoms of BPPV?

- I am dizzy when I roll over or move my head in certain ways
- The sensation I experience lasts only for a few minutes, but the nausea can last longer
- The room spins when I lie down

## Complete Instructional DVD

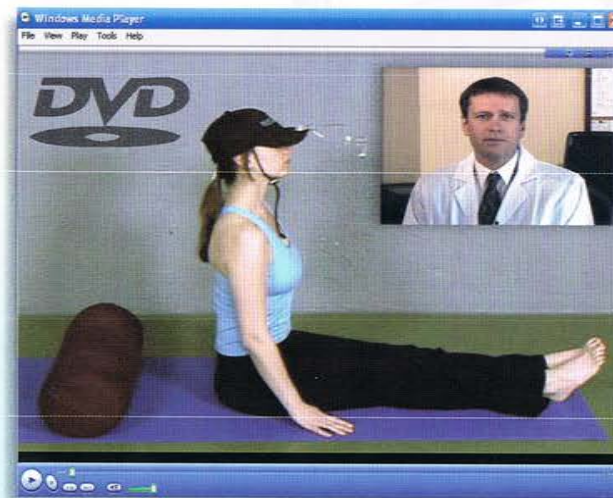

## The DizzyFIX Includes:

- DizzyFIX Device
- DizzyFIX Hat
- Written Instructions
- Instructional DVD
- Quick Reference Card

